

Nasal Decongester

Vital Baby® Nasal Decongester has been specially designed to clear baby's blocked nose quickly and effectively with a gentle squeeze. Phthalate, Latex and BPA free.

Before using your Vital Baby® Nasal Decongester, please ensure that you read the instructions carefully. For use on babies noses from 0+ months. Safe for microwave, steam and cold water sterilisation.



PARTS LIST

New Born Insertion Limit

Toddler Insertion Limit

Base Lip



SOFT SILICONE TIP



EASY-GRIP WINGS



COLLAR



BULB BASE

3 SIMPLE STEPS

Before each use, inspect the Nasal Decongester and all its components to ensure that they are clean and undamaged.

1. Holding the decongester as shown push the bottom of the bulb base inwards with your thumb (see diagram i). For best results hold the nasal decongester as shown, pushing the base with your thumb rather than squeezing the sides.



Diagram i

2. Whilst holding the base inwards with your thumb (see diagram i), gently place tip of the decongester into baby's nostril and tilt it to the side (see diagram ii). Be careful not to over-insert the decongester. Do not push past the relevant insertion limit.

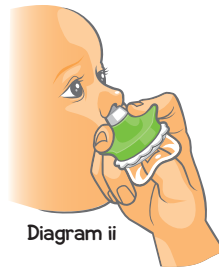


Diagram ii




Diagram iii

3. Slowly release the bulb allowing the suction to draw the mucus out of baby's nostril (see diagram iii).


CLEANING, CARE AND ASSEMBLY

Before using for the first time and after each use disassemble all components, wash with warm soapy water and sterilise according to your steriliser manufacturer's instructions.

- a) Loosen the Silicone Tip from the underside of the wings (see diagram a)

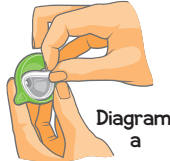


- b) Grip the Silicone Tip and gently pull upwards whilst moving from side to side until Tip comes out of Easy-Grip Wings (see diagram b).

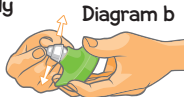


Wash all components with warm soapy water and sterilise according to your steriliser manufacturer's instructions. After cleaning and sterilising ensure you reassemble the Nasal Decongester correctly.

Push the Silicone Tip through the top of the Easy-Grip Wings (see diagram c).

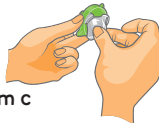

- c) Pull the Tip through the top of the Easy-Grip Wings until the Toddler Insertion Limit pops through and the Tip Lip is flush against inside of the Easy-Grip Wings (see diagram d).

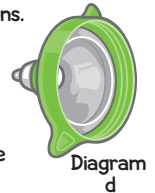

- d) Place the Collar around the top of the Bulb Base. Squeeze until the Base Lip fits around the Collar (see diagram e).



- e) Screw the Collar and Bulb Base to the Easy-Grip Wings (see diagram f).



WARNING for your child's safety and health:

- This product is not a toy. Adult supervision required.
- NEVER use the Nasal Decongester to force air into baby's nostril.
- Keep all components not in use out of reach of children.
- Be careful not to over-insert the decongester.
- Do not push past the over insertion limit.
- Do not allow the product to come into contact with oil-based foods (e.g. oil / tomato based sauces) as staining will occur.
- Clean all components as soon as possible after use.

Also available in the range:

Our Vital Baby® Aquaint® is a revolutionary 100% natural cleansing water that kills 99.9% of bacteria in seconds. It contains only 2 ingredients, Hypochlorous acid produced in the human body to fight bacteria and water to provide a base for it to be a highly effective cleanser. Can be used safely on the body from birth and on hard surfaces. Perfect to sanitise soothers, teething and feeding equipment when out and about - no need to rinse before giving to your baby.

